IMPORTANT: This is a single user product that should not be shared between moms. Use by more than one person may present a health risk, affect product performance and voids warranty.

This is a personal care product and therefore cannot be returned to the retailer once opened. If you require personal assistance with your pump or have lost or broken parts, please contact our Customer Service Department.

Simplisse, Incorporated

St. Louis, Missouri USA

Customer Service Department

Toll Free: 877-962-2525

M-F 8:00 am - 4:30 pm Central Time

email: support@simplisse.com

simplisse.com

© 2011 Simplisse, Inc.

Printed in the USA

U.S. Patent No. 7,972,297.

Manufactured by Simplisse, Incorporated, including parts from the US and China.

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IMPORTANT SAFEGUARDS

This is a single user product that should not be shared between moms. Use by more than one person may present a health risk, affect product performance and voids warranty.

When using electrical products, especially when children are present, the following basic safety precautions should always be followed.

Indications for Use: A personal use electric powered device used to express milk from the breast of a lactating woman.

READ ALL INSTRUCTIONS BEFORE USE

DANGER: To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store this product where it can fall or be pulled into a tub or sink.
- Do not place in or drop this product into water or other liquid.
- Do not reach for an electrical product if it has fallen into water. Unplug immediately.

WARNING: To reduce the risk of burns, electrocution, fire, or injury to persons:

- This product should never be left unattended when plugged into an electrical outlet.
- Close supervision is necessary when this product is used near children.
- Use this product only for its intended use as described in this manual.
- Do not use attachments not recommended by the manufacturer.

- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep the cord away from heated surfaces.
- Do not use outdoors, or operate where aerosol (spray) products are being used, or where oxygen is being administered.
- Plug the AC Power Cord into the product first and then into electrical outlet.
- To maintain compliance with the safety standard IEC 60601-1 only use the Simplisse Breast Pump with the AC Power Cord supplied
- Never use while sleepy or when drowsy.
- Never drop or insert any object into any opening or air tubing.
- The product has no user serviceable parts.

SAVE THESE INSTRUCTIONS

CLEANING

Make sure all Collection Unit parts have been disassembled for cleaning prior to use. Do not wash the **AC Power Cord**. For **Motor Unit**, wipe clean with dry or damp cloth as necessary. For **Air Tubing**, wipe clean as necessary. Refer to page 10 for additional cleaning information.

Refer to the Technical Information on the back cover for appropriate storage conditions.

THE SIMPLISSE® STORY

The Simplisse® Double Electric Breast Pump is a whole new way to think about expressing breastmilk. It was designed to feel like breastfeeding. It features a new Compression Technology that provides a gentle suction, similar to that of baby. It also has super-soft BabyFace® Flexible Breastcups that conform and naturally move with mom.

Naturally Better. By Design.®

TABLE OF CONTENTS:

Parts Diagram	4
Quick Start Assembly	5-7
Quick Start Expressing	7-8
Disassembly	9
Cleaning	10
Assembly	10-12
Expressing Breastmilk	12-14
Storing Breastmilk	14
Preparing Breastmilk	14
Tips	15
Troubleshooting	16

PARTS DIAGRAM

- A. Stand
- B. Collection Bottle
- C. Bottle Seal
- D. Connector
- E. Outer Breastcup
- F. Inner Breastcup
- G. Sealing Insert
- H. Cap
- I. Air Tubing
- J. Motor Unit
- K. AC Power Cord
- L. Air Plug

IMPORTANT NOTE TO SIMPLISSE MOMS: If you are a mom who is switching from a traditional breast pump, you may experience a short transition period while your breasts adjust to the compression technology of the Simplisse® Double Electric Breast Pump. This transition period is typically a matter of a few days, and you will still achieve milk flow. But if your milk volume seems slower at first, stick with it and you will see your milk volume return. This will not impact moms who are using an electric pump for the first time.

This section serves as a quick-start reference for Simplisse moms that are experienced with the Simplisse® Breast Pump. It is not a replacement for the complete instructions provided in this Instruction Booklet. Please be sure to read these thoroughly before expressing for the first time.

Before your first use and after all other uses, it is important to wash or sterilize your Breast Pump. Be sure to follow the complete directions in the Cleaning section of this Instruction Booklet.

ASSEMBLY

After cleaning, follow these steps to assemble your Collection Units:

1 Begin by placing the Bottle Seal into the bottom of the Connector.

2 Then secure the Collection Bottle to the Connector. Tightening the Collection Bottle firmly will seal the Bottle Seal, but do not over tighten.

3 Install the Inner Breastcup by pushing it firmly into the Connector opening.

4 To install the Outer Breastcup, first make sure the white ribs are aligned with the Inner Breastcup. Then push at the edges to secure the Outer Breastcup to the Connector.

MOTOR UNIT ASSEMBLY

Quick Start ▶

5

Make sure the Outer Breastcup is sealed all around. This is important for the pump to function properly.

Note: Run your fingers and thumbs around the edge to make sure the Outer Breastcup is affixed all around.

6

To prepare the Cap, first connect the Air Tubing to the top of the Cap using the end with two long prongs and one short one. This only fits one way. Make sure the Air Tubing is pushed in firmly to ensure an airtight seal.

7

Then take the Sealing Insert and push it on to the large ring on the underside of the Cap.

Note: Make sure the tab of the Sealing Insert is facing out.

8

Orient the two holes in the Sealing Insert around the two long prongs.

9

Install the Cap assembly to the Connector assembly by pivoting the front latch to the Connector.

Note: Make sure all parts are assembled appropriately and snugly fit. An air tight Collection Unit is necessary to function properly.

10

Pull down on the back portion to snap the tab on the Connector.

Then follow these steps to prepare the Motor Unit and Air Tubing:

11

Place the finished Collection Unit into one of the stands. Repeat these steps to assemble the second Collection Unit.

12

Place the Motor Unit on a sturdy table or desk. Insert the AC Power Cord into the Motor Unit, then plug into your electrical outlet.

13

Then insert the 3-prong end of the Air Tubing into the Motor Unit. It can only be installed in one orientation. Repeat Step 13 for the second Collection Unit.

EXPRESSING BREASTMILK

Our Simplisse® Breast Pump is a whole new way to think about expressing breastmilk. It features a new compression technology and super-soft flexible breastcups.

Once your Collection Unit is cleaned and assembled, you are ready to begin expressing. Our Breast Pump allows you to express from both breasts or from one breast.

- 1 Make sure your hands and breasts are clean.
- 2 Make sure the Motor Unit is plugged in, and make sure that the Motor Unit, Air Tubing, and Collection Units are connected properly.

3

Double Expressing: For Double Expressing, make sure both strands of air tubing are securely connected to the Motor Unit.

Single Expressing: For Single Expressing, insert one strand of air tubing in either the left side or right side. Close the other side by inserting the Air Plug.

Note: Inserting the Air Plug is necessary for the pump to function properly while Single Expressing. Make sure this is air tight.

4

Turn the Motor Unit on by pushing the Power Button.

5

Dial Settings: The **speed dial** controls how fast the pump cycles. The **flow dial** controls the compression and vacuum level at the breasts.

Every mom is different and therefore the right combination of speed and flow settings will be an individual choice based on comfort and volume of milk flow. However, the following is a general guideline to get you started.

When starting your expressing session, turn the flow dial to C and the speed dial to 10. This is the setting most likely to help you achieve letdown (milkflow). Once your milk begins to flow, turn the speed dial to 5 for a slower rhythm. Be sure to experiment with different dial settings to maximize your milk flow and comfort.

You may hear an airy, wispy noise. This comes from the Valve and is completely normal.

Note: The Simplisse Breast Pump is designed to automatically shut off after 30 minutes.

6

Apply the Collection Units to your breasts, one at a time. Position the Breastcups, nipple centered, firm enough to ensure a seal with no air leaks.

Note: Be sure to always hold your Collection Units level when expressing. This ensures that your Breast Pump performs as intended.

7

Then position the Breastcup of the second Collection Unit.

8 Once you have achieved Let-Down, adjust the Flow and Speed dials as necessary until they are comfortable for you. **Let-Down is the biological trigger to release breastmilk and it's the first step for a successful pumping session.**

9 Breastmilk should begin to flow and collect in the bottle. Even if you do not experience milk flow immediately, continue expressing for a few minutes. Your first few times expressing may not yield any milk. This is normal. Never exceed 5 minutes of expressing without milk flow.

10 After expressing, be sure to place the filled Collection Units in the stands. This keeps them stable and helps prevent spilling, and allows you free hands to turn off the Motor Unit and unscrew the Collection Bottles. Always clean or sterilize your Collection Unit parts after every expressing session.

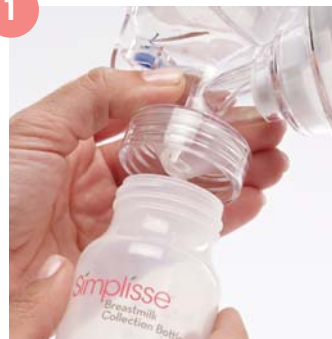
WARNING: Expressing milk should not cause pain or discomfort. If you experience pain or discomfort during expressing, stop using the device immediately. The suction can be broken by inserting your finger between your breast and the outer breastcup. You can also break the suction by unscrewing the Collection Bottle. To turn off the device, push the power button or unplug the Power Cord from the Motor Unit.

Quick Start ■

DISASSEMBLY

Follow these steps to disassemble the Collection Units for cleaning:

1

First loosen and remove the Collection Bottle from the Connector.

2

Then remove the Bottle Seal from the underside of the Connector.

3

To remove the Outer Breastcup, gently pull on the perimeter tabs to separate it from the Connector.

4

To remove the Inner Breastcup, use your finger to gently pull it loose from the Connector.

5

To remove the Cap, simply pull up on the Cap tab at the back. This will separate from the Connector.

6

With the Cap in hand, pull the Sealing Insert downward to dislodge.

Note: The Simplisse Breast Pump has a small valve in the Connector. Do not remove this valve. This is a permanent part.

CLEANING

Before your first use and after all other uses, it is important to wash or sterilize your Breast Pump:

Make sure all Collection Unit parts have been disassembled for cleaning prior to use. Do not wash the **AC Power Cord**. For **Motor Unit**, wipe clean with dry or damp cloth as necessary.

For **Air Tubing**, wipe exterior clean with dry or damp cloth as necessary. Do not get interior of Air Tubing wet. Do not submerge Air Tubing in water. Do not sterilize Air Tubing.

For Collection Unit parts:

TO HAND WASH: Wash all parts in warm, soapy water, rinse well and air dry.

TO STEAM STERILIZE: Use a microwave steam sterilizer or steam sterilizer bags. Be sure to follow the instructions that come with your sterilizer or steam bags.

IN DISHWASHER: All pump parts are top-rack dishwasher safe. Completely air dry parts and let them cool.

TO BOIL STERILIZE: Boil parts for 5 minutes in an open pot. Completely air dry parts and let them cool.

The **Inner Breastcup**, **Outer Breastcup** and **Sealing Insert** are delicate parts. Please take extra care in cleaning them. It is recommended that you hand wash these parts. Do not use any rough brushes to clean these parts.

Note: If the shape of your Inner Breastcup has changed slightly after cleaning, don't worry. It will fit properly into the Connector and resume its shape.

ASSEMBLY

After cleaning, follow these steps to assemble your Collection Units:

Begin by placing the Bottle Seal into the bottom of the Connector.

Then secure the Collection Bottle to the Connector. Tightening the Collection Bottle firmly will seal the Bottle Seal, but do not over tighten.

Install the Inner Breastcup by pushing it firmly into the Connector opening.

To install the Outer Breastcup, first make sure the white ribs are aligned with the Inner Breastcup. Then push at the edges to secure Outer Breastcup to the Connector.

Make sure the Outer Breastcup is sealed all around. This is important for the pump to function properly.

Note: Run your fingers and thumbs around the edge to make sure the Outer Breastcup is affixed all around.

To prepare the Cap, first connect the Air Tubing to the top of the Cap using the end with two long prongs and one short one. This only fits one way. Make sure the Air Tubing is pushed in firmly to ensure an airtight seal.

Then take the Sealing Insert and push it on to the large ring on the underside of the Cap.

Note: Make sure the tab of the Sealing Insert is facing out.

Orient the two holes in the Sealing Insert around the two long prongs.

Install the Cap assembly to the Connector assembly by pivoting the front latch to the Connector.

Pull down on the back portion to snap the tab on the Connector.

Note: Make sure all parts are assembled appropriately and snugly fit. An air tight Collection Unit is necessary to function properly.

Then follow these steps to prepare the Motor Unit and Air Tubing:

11

Place the finished Collection Unit into one of the stands. Repeat these steps to assemble the second Collection Unit.

12

Place the Motor Unit on a sturdy table or desk. Insert the AC Power Cord into the Motor Unit, then plug into your electrical outlet.

13

Then insert the 3-prong end of the Air Tubing into the Motor Unit. It can only be installed in one orientation. Repeat Step 13 for the second Collection Unit.

EXPRESSING BREASTMILK

Our Simplisse® Breast Pump is a whole new way to think about expressing breastmilk. It features a new compression technology and super-soft flexible breastcups.

Once your Collection Unit is cleaned and assembled, you are ready to begin expressing. Our Breast Pump allows you to express from both breasts or from one breast.

- 1 Make sure your hands and breasts are clean.
- 2 Make sure the Motor Unit is plugged in, and make sure that the Motor Unit, Air Tubing, and Collection Units are connected properly.

3

Double Expressing:

For Double Expressing, make sure both strands of air tubing are securely connected to the Motor Unit.

Single Expressing:

For Single Expressing, insert one strand of air tubing in either the left side or right side. Close the other side by inserting the Air Plug.

Note: Inserting the Air Plug is necessary for the pump to function properly while Single Expressing. Make sure this is air tight.

4

Turn the Motor Unit on by pushing the Power Button.

5

Dial Settings: The **speed dial** controls how fast the pump cycles. The **flow dial** controls the compression and vacuum level at the breastcups.

Every mom is different and therefore the right combination of speed and flow settings will be an individual choice based on comfort and volume of milk flow. However, the following is a general guideline to get you started.

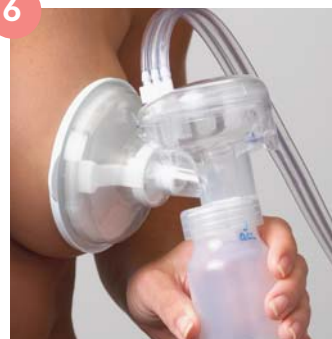
When starting your expressing session, turn the flow dial to C and the speed dial to 10. This is the setting most likely to help you achieve letdown (milkflow). Once your milk begins to flow, turn the speed dial to 5 for a slower rhythm. Be sure to experiment with different dial settings to maximize your milk flow and comfort.

You may hear an airy, wispy noise. This comes from the Valve and is completely normal.

Note: The Simplisse Breast Pump is designed to automatically shut off after 30 minutes.

6

Apply the Collection Units to your breasts, one at a time. Position the Breastcups, nipple centered, firm enough to ensure a seal with no air leaks.

7

Then position the Breastcup of the second Collection Unit.

Note: Be sure to always hold your Collection Units level when expressing. This ensures that your Breast Pump performs as intended.

8

Once you have achieved Let-Down, adjust the Flow and Speed dials as necessary until they are comfortable for you. **Let-Down is the biological trigger to release breastmilk and it's the first step for a successful pumping session.**

9 Breastmilk should begin to flow and collect in the bottle. Even if you do not experience milk flow immediately, continue expressing for a few minutes. Your first few times expressing may not yield any milk. This is normal. Never exceed 5 minutes of expressing without milk flow.

10 After expressing, be sure to place the filled Collection Units in the stands. This keeps them stable and helps prevent spilling, and allows you free hands to turn off the Motor Unit and unscrew the Collection Bottles. Always clean or sterilize your Collection Unit parts after every expressing session.

WARNING: Expressing milk should not cause pain or discomfort. If you experience pain or discomfort during expressing, stop using the device immediately. The suction can be broken by inserting your finger between your breast and the outer breastcup. You can also break the suction by unscrewing the collection bottle. To turn off the device, push the power button or unplug the power cord from the Motor Unit.

STORING & HANDLING BREASTMILK

After expressing your breastmilk, it is important to store and handle it properly.

- Make sure your hands and storage containers are clean to avoid contamination of your expressed breastmilk.
- Use Breastmilk Collection Bottles or Breastmilk Storage Bags to store your breastmilk – these can be used in the refrigerator or freezer.
- Be sure your storage bottle or bag is sealed tightly to ensure freshness of the breastmilk.
- Date code your bottles or bags if you do not intend to use your collected breastmilk immediately.
- For extended storage, freeze your breastmilk.

WARNING: Do not add fresh breastmilk to already frozen breastmilk in a storage container. Do not save breastmilk from a used bottle for another feeding.

THAWING & PREPARING BREASTMILK

After storing your breastmilk, follow these steps to prepare it for baby.

WARNING: Do not microwave breastmilk. Do not microwave Collection Bottles or Storage Bags containing breastmilk. Microwave ovens can create uneven heating which could scald baby and deteriorate the nutrients of breastmilk.

- To thaw breastmilk, place the sealed Collection Bottle or Storage Bag in refrigerator or run under cool tap water.
- To warm breastmilk, place the sealed Collection Bottle or Storage Bag in warm tap water.
- Always check breastmilk temperature before feeding baby.
- Use thawed breastmilk immediately or refrigerate up to 24 hours.

Note: Do not thaw breastmilk in very hot or boiling water. Once thawed, do not refreeze or refrigerate breastmilk again. Do not allow thawed breastmilk to sit at room temperature.

Storage Duration of Fresh Breastmilk (For Use with Healthy Full Term Infants)

LOCATION	TEMPERATURE	DURATION
Countertop, table	Room temperature (up to 77°F or 25°C) <i>Containers should be covered and kept as cool as possible; covering the container with a cool towel may keep milk cooler</i>	6-8 hours
Insulated cooler bag	5-39°F or -15-4°C <i>Keep ice packs in contact with milk containers at all times, limit opening cooler bag.</i>	24 hours
Refrigerator	39°F or 4°C <i>Store milk in the back of the main body of the refrigerator.</i>	5 days
Freezer compartment of a refrigerator	5°F or -15°C	2 weeks
Freezer compartment of refrigerator with separate doors	-0°F or -18°C	3-6 months
Chest or upright deep freezer	-4°F or -20°C <i>Store milk toward the back of the freezer, where temperature is most constant. Milk stored for longer durations in the ranges listed is safe, but some of the lipids in the milk undergo degradation resulting in lower quality.</i>	6-12 months

Reference: Academy of Breastfeeding Medicine. (2004) Clinical Protocol Number #8: Human Milk Storage Information for Home Use for Healthy Full Term Infants. Princeton Junction, New Jersey: Academy of Breastfeeding Medicine.

TIPS

Breastfeeding and expressing are unique experiences for nursing moms. Please review the suggestions below.

- Moms are more likely to successfully express if they are relaxed and thinking about baby. Avoid expressing if you are rushed.
- Efficient expressing usually takes patience and practice for moms. Moms that express frequently become accustomed to it and are more likely to have faster Let-Down and more milk flow.
- All moms are different. Some experience Let-Down quickly and some moms do not. Some express on their first try and some moms do not.
- Do not be discouraged if you are unable to express milk on your first try. Simply take a break and try again later.
- Do not be discouraged if you express only a small amount of breastmilk. This is common especially on the first few attempts.
- Expressing consistently, in addition to breastfeeding, can help sustain your milk supply over time.
- A warm compress or a warm bath/shower can help encourage milk flow.
- **WARNING:** Expressing milk should not cause pain or discomfort. If you experience pain or discomfort during expressing, stop using the device immediately. The suction can be broken by inserting your finger between your breast and the outer breastcup. You can also break the suction by unscrewing the collection bottle. To turn off the device, push the power button or unplug the power cord from the Motor Unit.

TROUBLESHOOTING

- For your Breast Pump to work as designed, all parts must be properly assembled. Make sure all parts are snugly fit so you have an air tight Collection Unit.
- When applying to your breast, make sure the Breastcup is centered around your nipple. Hold the Collection Unit firm enough to your breast for a complete seal all around, but not too firm as this could inhibit milk flow.
- When the Breastcup is applied, make sure that you feel a light suction on your breast.
- Be sure to always hold your Collection Units level when expressing. This ensures that your Breast Pump performs as intended.
- If you hear an airy, wispy noise as you express, it is completely normal.
- The Simplisse Breast Pump is designed to automatically shut off after 30 minutes.
- If you require personal assistance or have lost or broken parts, please call our Simplisse Customer Service Department at 877-962-2525 or support@simplisse.com.

Technical Information

Environmental Conditions (Transport and Storage)	
Temperature	-4°F to 176°F (-20°C to 80°C)
Relative Humidity	10%-90% non-condensing
Environmental Conditions (Operation)	
Temperature	34°F to 95°F (0°C to 35°C)
Relative Humidity	10%-90% non-condensing
Classification	
<input type="checkbox"/>	Class II
<input checked="" type="checkbox"/>	Type BF
Definitions	
	Attention, consult accompanying documents

This product is for household use.

Electromagnetic or Other Interference:

This device complies with IEC 60601-1-2 and is unlikely to interfere with other electrical equipment. This device may emit RF energy; if interfering with other devices, relocate.

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Référence : Academy of Breastfeeding Medicine. (2004) Clinical Protocol Number #8: Human Milk Storage Information for Home Use for Healthy Full Term Infants. Princeton Junction, New Jersey: Academy of Breastfeeding Medicine.

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WARRANTY POLICY

The Simplisse® Breast Pump is a single-user product. Use by more than one person voids this warranty.

Simplisse, Inc. warrants this product for one year from the date of purchase for the pump motor. Parts and accessories are warranted for 90 days from the date of purchase. If the product is found to be defective, Simplisse will replace the product or part free of charge. Any misuse, abuse or alterations voids this warranty.

Returns are not accepted without an authorization number. To obtain an authorization number, contact Simplisse Customer Service at www.simplisse.com or toll-free at 877-962-2525. You can then return the product with the authorization number, proof of purchase, and explanation of the problem to:

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